

BOLIN™

A Product of MBI, Inc.

X-RAY SHIELD STAND

Assembly & Installation Instructions

1. Carefully unpack your new *Bolin Shield Stand* column leaving the shrink wrap around column, holding the arm slides in place.
2. Lay it down on a clean and flat surface to access bottom cover plate (fig. 1-D)
3. Remove the (4) screws holding the bottom cover plate (fig. 1-D).
4. Remove the foam packing material from inside the (4) tubes. (Fig. 2-H)
5. Reinstall the bottom cover plate with the (4) screws removed in step 3.
6. Remove the (4) screws and washers from the top of the column being careful not to disturb the bearing plate.
7. Locate the top mounting bracket (fig. 3) and mount it to the top of the column with the (4) screws and washers removed in step 6.
8. Stand the column up vertically and remove the shrink wrap while holding the top arm, these slides are counter weighted so **BE CAREFUL** as you release them.
9. Find the best position for the shield stand (fig. 1-G) and secure it to both the floor and wall with the appropriate hardware (not supplied).
10. Carefully unpack the shield arms and x-ray shields (fig. 4).
11. Unscrew the Allen head shoulder screws in each of the arm slides (fig. 1-C).
12. Insert arm into arm slide, reinstall and hand tighten Allen head shoulder screw (do not over tighten these screws).
13. Insert X-Ray shields starting at the Bottom, 1-Gonad shield Male or Female, 2 –Breast shield, 3-Thyroid shield, 4 Eye/Brain shield, (fig. 4).

Rev.1

Designed for convenience... without compromising safety.

BOLIN X-Ray Products · A Division of MBI, Inc. · (702) 259-1999 · Fax (702) 259-1090

BOLIN™

A Product of MBI, Inc.

Rev.1

